MOPEDIA®

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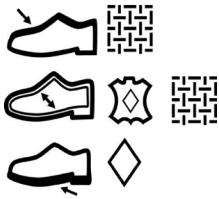
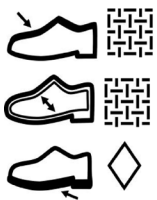
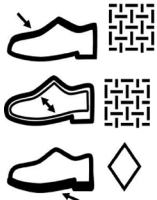
12. CARATTERISTICHE TECNICHE

Tipologia	Codici prodotto	Taglia	Misure	Materiale
Calzature post-operatorie	SC100-XS	XS	35/36	
	SC100-S	S	37/38	
	SC100-M	M	39/40	
	SC100-L	L	41/42	
	SC100-XL	XL	43/44	
	SC100-XXL	XXL	45/46	
	SC110-XS	XS	35/36	
	SC110-S	S	37/38	
	SC110-M	M	39/40	
	SC110-L	L	41/42	
	SC110-XL	XL	43/44	
	SC110-XXL	XXL	45/46	
	SC120-XS	XS	35/36	
	SC120-S	S	37/38	
SC120-M	M	39/40		
SC120-L	L	41/42		
SC120-XL	XL	43/44		
SC120-XXL	XXL	45/46		

Tipologia	Codici prodotto	 L:	Misure	Materiale	
Calazature fisiologiche	SC200-35	23,1 cm*	35		
	SC200-36	23,7 cm*	36		
	SC200-37	24,4 cm*	37		
	SC200-38	25,1 cm*	38		
	SC200-39	25,7 cm*	39		
	SC200-40	26,4 cm*	40		
	SC200-41	27,1 cm*	41		
	SC200-42	27,8 cm*	42		
	SC200-43	28,4 cm*	43		
	SC200-44	29,1 cm*	44		
	SC200-45	29,8 cm*	45		
	Calazature fisiologiche	SC210-35	23,1 cm*	35	
		SC210-36	23,7 cm*	36	
		SC210-37	24,4 cm*	37	
		SC210-38	25,1 cm*	38	
		SC210-39	25,7 cm*	39	
		SC210-40	26,4 cm*	40	
		SC210-41	27,1 cm*	41	
		SC210-42	27,8 cm*	42	
		SC210-43	28,4 cm*	43	
SC210-44		29,1 cm*	44		
SC210-45	29,8 cm*	45			
Tipologia	Codici prodotto	Taglia	Misure	Materiale	
Coprigesso	SC300-XS	XS	35/36		
	SC300-S	S	37/38		
	SC300-M	M	39/40		
	SC300-L	L	41/42		
	SC300-XL	XL	43/44		
	SC300-XXL	XXL	45/46		

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MOPEDIA®

MA SC100-xx_SC110-xx_SC120-xx_SC200-xx_SC210-xx_SC300-xx 01 A*_ENG_10-2023

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1. CODE

SC100-xx	Open toe shoe with hard sole
SC110-xx	Baruk-type shoe
SC120-xx	Full opening shoe - rigid sole
SC200-xx	Elasticated physiological shoe
SC210-xx	Full opening physiological shoe
SC300-xx	Coverslip

2. INTRODUCTION

Thank you for choosing a product from the MOPEDIA line. MOPEDIA orthopedic footwear is designed and manufactured to meet all your needs for practical, correct and safe use. This manual contains small tips for proper use of your chosen device and valuable advice for your safety. We recommend that you read the entirety of this manual carefully before using the device. If in doubt, please contact your dealer, who will be able to help and advise you correctly.

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3. INTENDED USE

MOPEDIA Orthopedic Footwear is indicated for foot protection as part of conservative, postoperative and/or preventive treatment.

WARNING!

The choice of shoe model should be made only by a specializing technician or a qualified and licensed professional.

4. DECLARATION OF CONFORMITY

Moretti S.p.A. declares under its sole responsibility that the product made and traded by Moretti S.p.A. and belonging to the group of ORTHOPEDIC FOOTWEARS - MOPEDIA complies with the provisions of the regulation 2017/745 on MEDICAL DEVICES of 5 April 2017. For this purpose, Moretti S.p.A. guarantees and declares under its sole responsibility what follows:

1. The devices satisfy the requirements of general safety and performance requested by the Annex I of regulation 2017/745 as laid down by the Annex IV of the above mentioned regulation.
2. The devices ARE NOT MEASURING INSTRUMENTS.
3. The devices ARE NOT MADE FOR CLINICAL TESTS.
4. The devices are packed in NON-STERILE BOX.
5. The devices belong to class I in accordance with the provisions of Annex VIII of the above mentioned regulation
6. Moretti S.p.A. provides to the Competent Authorities the technical documentation to prove the conformity to the 2017/745 regulation, for at least 10 years from the last lot production.

Note: Complete product codes, the manufacturer registration code (SRN), the UDI-DI code and any references to used regulations are included in the EU declaration of conformity that Moretti S.p.A. releases and makes available through its channels.

5. GENERAL WARNINGS

- Refer to this manual for correct use of the device
- Always consult your doctor or therapist for proper use of the device
- Check that the product is delivered to you undamaged and keep it packed away from heat sources
- The life of the device is determined by the wear and tear of non-repairable and/or replaceable parts
- Always pay attention to the presence of children
- The manufacturer and retailer accept no liability for damage caused to persons, animals or property resulting from intervention by unauthorised personnel and failure to observe of this instruction manual
- The user and/or patient must report any serious incidents occurring in connection with the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is established.
- Use the device after evaluation by a qualified technician, following its instructions for use and those given in this manual.

WARNING!

Do not let skin come in direct contact with footwear, always wear a sock of wraps, or a combination of both.

6. SYMBOLS

Product code

Unique Device Identification

CE mark

Manufacturer

Batch Lot

Read the instruction manual

Medical Device

Single patient multiple use

Do not bleach

Wash by hand; Maximum temperature 30°C

Do not dry clean

Do not tumble dry

Do not iron

7. DESCRIPTION AND GENERAL VIEW

Series SC100

- Adjustable velcro laces
- Removable top
- Rigid back
- Ambidextrous

Series SC110

- Adjustable and removable Velcro strap
- Rigid counter
- Workable sole
- mbidextrous

Nota: The workmanship should only be performed by a licensed orthopedic technician after evaluation

Series SC120

- Shoe with adjustable and fully openable closures
- Rigid sole
- Ambidextrous

Series SC200

- Shoe with Velcro closure
- Stiff, reinforced counter
- Elastic and adaptable front fabric
- Right and left

Series SC210

- Shoe with adjustable Velcro closures
- Adjustable back opening
- Right and left

Series SC300

- Ambidextrous

7.1 Indications for use post operative shoes

The Series SC100, SC110, SC120 postoperative footwear protects the foot following surgery by providing plenty of space for bandages and can be opened widely. In addition, the foot is stabilized and locked.

SC100

- Fully open the footwear;
- Insert the foot protected by sock or bandage;
- Depending on the arrangement of the specialists, remove or not remove the upper flap;
- Close the sock without over-tightening the Velcro laces;
- Make sure the foot is well supported, or that it is not too loose before walking.

SC110

- Open the footwear completely;
- Insert foot protected by sock or bandage;
- Close the shoe without over-tightening the Velcro laces;
- Make sure the foot is well supported, or that it is not too loose before walking.

SC120

- Open the footwear completely;
- Insert foot protected by sock or bandage;
- Close the shoe without over-tightening the Velcro laces;
- Make sure the foot is well supported, or not too loose before walking.

7.2 indication for use physiological shoes

Series SC200, SC210 physiological shoes are preventive shoes to facilitate the management of a mildly compromised foot, preventing the aggravation of pathological conditions, and provide comfort and ergonomics in the home environment.

SC200

- Open the shoe completely;
- Make sure to fit the shoe correctly and close the Velcro strap firmly, but without over-tightening.

SC210

- Open the shoe completely;
- Insert foot protected by sock or bandage;
- Close the shoe starting from the top by adjusting it as needed and then then close the back part firmly;
- Make sure the foot is well supported, or that it is not too loose before walking. .

7.3 Indication for use shoe cover

SC300

- Open the shoe completely;
- Gently insert the bandaged or plastered foot;
- Be sure to fasten the laces securely, without overtightening.

8. BEFORE USE

- Always check the state of wear and cleanliness of parts so as to avoid infection;
- Ensure that stockings and bandages are properly worn, to avoid thicknesses from folded tissues;
- Make sure you have chosen the appropriate size of footwear;
- Wear a strictly clean sock, bandage or combination thereof;
- Check for the absence of debris inside the shoe.

9. WARNINGS FOR USE

- Be sure to walk on flat, uneven surfaces if wearing a post-operative shoe;
- Climbing and descending stairs can be risky; make sure you have a handrail, or do so with the help of medical personnel;
- Make sure together with the medical professional that you have chosen the correct model of shoe in relation to the specific needs;

Note: Mopedia's line of orthopedic footwear, is designed and commensurate for use in a controlled, home environment; they are not suitable for outdoor (occasional or prolonged) use.

WARNING!

Possibility of traffic accidents due to limited driving ability;

Do not ride a bicycle or drive any other type of vehicle while wearing a therapeutic shoe.

WARNING!

Danger of falling due to inappropriate use;

Inquire with your orthopedic technician or a qualified/licensed professional regarding proper use of the shoe, possible hazards (e.g., slippery and wet surface), as well as hazards regarding your indication and possibly resulting physical limitations.

10. CLEANING AND DISINFECTION

10.1 Cleaning

Washing indication:

- Keep the shoe clean by regularly removing dirt and debris from the sole and fabric;
- Inspect it regularly externally and internally;
- Maintain good overall clean conditions;
- Try to avoid external contact with dirt of a biological nature, in case clean and disinfect the shoe immediately;
- The therapeutic shoe should not be dried in a dryer or with direct heat sources direct: let it dry slowly at room temperature;
- It is recommended to leave the footwear open after wearing them, to prevent the bacterial proliferation;
- Be sure to wear clean socks or bandages with each use.
- Assicurarasi di indossare calze o fasciature pulite ad ogni utilizzo.

10.2 Disinfection

Avoid solvents that can damage fabrics; use neutral soaps and disinfectants.

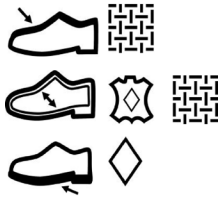
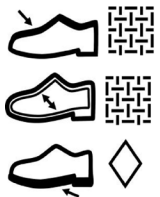
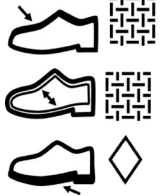
Nota: Absolutely avoid using acidic, alkaline or solvent products such as acetone or thinner

11. CONDITIONS OF DISPOSAL

Solid urban waste. Instead, it is recommended that the device be disposed of through the municipal recycling centres for the planned recycling of the materials used.

12. TECHNICAL FEATURES

Type	Product code	Size	Measure	Material
Post-operative shoes	SC100-XS	XS	35/36	
	SC100-S	S	37/38	
	SC100-M	M	39/40	
	SC100-L	L	41/42	
	SC100-XL	XL	43/44	
	SC100-XXL	XXL	45/46	
	SC110-XS	XS	35/36	
	SC110-S	S	37/38	
	SC110-M	M	39/40	
	SC110-L	L	41/42	
	SC110-XL	XL	43/44	
	SC110-XXL	XXL	45/46	
	SC120-XS	XS	35/36	
	SC120-S	S	37/38	
	SC120-M	M	39/40	
SC120-L	L	41/42		
SC120-XL	XL	43/44		
SC120-XXL	XXL	45/46		

Type	Product code	 L:	Measure	Material	
Physiological shoes	SC200-35	23,1 cm*	35		
	SC200-36	23,7 cm*	36		
	SC200-37	24,4 cm*	37		
	SC200-38	25,1 cm*	38		
	SC200-39	25,7 cm*	39		
	SC200-40	26,4 cm*	40		
	SC200-41	27,1 cm*	41		
	SC200-42	27,8 cm*	42		
	SC200-43	28,4 cm*	43		
	SC200-44	29,1 cm*	44		
	SC200-45	29,8 cm*	45		
	Physiological shoes	SC210-35	23,1 cm*	35	
		SC210-36	23,7 cm*	36	
		SC210-37	24,4 cm*	37	
		SC210-38	25,1 cm*	38	
		SC210-39	25,7 cm*	39	
		SC210-40	26,4 cm*	40	
		SC210-41	27,1 cm*	41	
		SC210-42	27,8 cm*	42	
		SC210-43	28,4 cm*	43	
SC210-44		29,1 cm*	44		
SC210-45	29,8 cm*	45			
Type	Product code	Size	Measure	Material	
Shoe cover	SC300-XS	XS	35/36		
	SC300-S	S	37/38		
	SC300-M	M	39/40		
	SC300-L	L	41/42		
	SC300-XL	XL	43/44		
	SC300-XXL	XXL	45/46		

Legenda materiale

		
Upper	Upper coating	Outsole
		
Coated leather	Natural textiles and non-woven synthetics	other materials

13. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for 2 years from the purchasing date, except possible exclusion or restriction as follows. The warranty shall not be apply in the event damages caused by improper use, abuse or alteration, and for the warranty will not be valid if the instructions for use are not strictly adhered to. The correct intended use is specified in this manual. Moretti is not responsible of consequent damages, personal injuries or whatever caused by or in relation with mistaken installation or with device improper use. Moretti does not guarantee for damages caused by the following condition: natural disaster, not authorized maintenance or repairs , faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Moretti, improper use, alteration not authorized,

13.1 Repairing

Warranty repair: If a Moretti item presents material or manufacturing faults during the warranty period, Moretti will estimate with customer if the fault can be cover from warranty. Moretti, at its unquestionable discretion, could repair or replace the item, with a Moretti dealer or with own base. Labor cost can be in charge to Moretti if the repair is cover from warranty. A repair or a replacement doesn't extend the warranty.

Repair not cover from warranty: A product without warranty can be given back after Moretti authorization. The labor and shipping costs for a bath lift out of warranty are to be payed by the customer or by the dealer. The repairs are guaranteed for 6 months as from the good received.

Non-defective device: The customer will be informed if, after the device return and examination, Moretti declares that the device is not faulty. In this case the good will be sent back to customer, the shipping charge will be to be payed by the customer.

13.2 Exempt clauses

Moretti doesn't offer any other declarations, warranty or conditions, explicit or implicit, including possible declarations, warranties or condition of merchantability, fitness for a specific purpose, non infringement and non interference, all but what expressly specified in this warranty. Moretti doesn't guarantee the nonstop and faultless usage. The duration of possible implicit warranties which can be fixed by according to the law is limited by the warranty period, in the limits of law. Some states or countries don't allow to limit the implicit warranty or the exclusion or the limitation for accidental damages. In such countries, some of those exclusions or limitation may not be applied to the user. The present warranty may be modified without prior notification.shipment damages (different from original Moretti shipment), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

WARRANTY CERTIFICATE

ENGLISH

Product _____

Purchased on (date) _____

Retailer _____

Address _____ Town/city _____

Sold to _____

Address _____ Town/city _____

Via Bruxelles, 3 - Melegnano 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

** Please consult our website for the latest available version of the user manual*

MOPEDIA®

MA SC100-xx_SC110-xx_SC120-xx_SC200-xx_SC210-xx_SC300-xx 01 A* ESP_10-2023

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